



KETTENBACH
Simply intelligent

Panasil®

For precise clear aligner impressions



1-877-532-2123

Panasil®

For precise clear aligner impressions



Panasil® putty soft and Panasil® contact plus will be used for this Invisalign impression



One scoop of Panasil® putty soft base and catalyst is enough for an upper full arch medium impression tray



Mix putty thoroughly with clean dry hands or using powderless gloves



Load the tray



Take your fingers and make a depression for the wash material



Dispense the wash material onto the putty



Insert into the mouth



Finished impression



Bite registration

Panasil® putty

- 11101 Panasil® putty
Normal pack 2 x 450mL, 2 scoops
- 11141 Panasil® putty fast set
Normal pack 2 x 450mL, 2 scoops
- 11121 Panasil® putty soft set
Normal pack 2 x 450mL, 2 scoops

Wash materials

- 13401 Panasil® initial contact x-light
Normal pack 2 x 50mL, 12 mixing tips
- 13411 Panasil® initial contact light
Normal pack 2 x 50mL, 12 mixing tips
- 13431 Panasil® initial contact regular
Normal pack 2 x 50mL, 12 mixing tips

Accessories:

- 17240 Mixing tips, yellow bag of 100
- 17222 Intraoral tips, yellow bag of 96
- 17203 Applyfix 4 dispenser



Additional viscosities available:

- Panasil® tray heavy
- Panasil® monophasic medium
- Panasil® binetics putty



www.kettenbach.com/www.kettenbachusa.com

Canadian orders, please contact Cerum at 1-800-661-9567
For U.S. orders buy direct from Kettenbach at:
7777 Center Avenue, Suite 280 • Huntington Beach, CA 92647
Toll free fax: 1-888-892-9820 • Local fax: 714-891-7860
Toll free phone: 1-877-532-2123 • Local phone: 714-892-9820