

Quick pick up. Dynamic impressions.

Panasil[®] monophasic medium

A-silicone specifically for transfer, pick-up and standard crown & bridge impressions.



Whether fixed, removable or implant-supported restorations, one thing is always true: The impression is of vital importance. Panasil monophasic medium is optimized for taking uncompromisingly precise impressions and, being a medium –viscosity monophasic impression material with initial hydrophilicity is especially advantageous for pick up of impressions for fixed, removable and implant supported impressions. Panasil monophasic medium exhibits a fast setting time, high final hardness and ideal elasticity. The flow properties have also been optimized as well as substantial thixotropic properties to ensure that there is pressure build up in the tray, but the material will still flow intimately into the sulcus, impression copings or metal frameworks. Panasil monophasic medium is available in the foil bag system for the Plug & Press Dispenser as well as in cartridges.

Panasil® monophasic medium — Advantages at a glance:

Easy to use in the dental office or in the laboratory

- The thixotropic properties will make sure of an immediate, tight flowing around the inner copings, impression posts or preparation margins.
- The high Shore A hardness retains the inner copings firmly and increases the precision of fit.
- Highly elastic to allow the impression to be removed from the mouth easily, with high tear-resistance and minimal risk of fracturing the model.
- The brief intraoral setting time and total working time tailored to the needs of dental practices make Panasil® monophasic medium especially easy to use so result in less distortion during setting and reduce the strain on the patient.
- The high thixotropicity builds up the pressure well.



Working and setting times:

Total working time: 2 minutes
 Intraoral setting time: 2 minutes
 Final set: 4 minutes

Panasil® monophasic medium

Medium viscosity, monophasic polyvinyl siloxane impression material

Panasil® monophasic medium

11846 Panasil® monophasic medium
 Normal Pack cartridges 2 x 50mL, 6 mix tips



Panasil® monophasic medium

28280 Panasil® monophasic medium
 Intro Kit pack for Plug & Press® Dispenser
 1 x 362mL, 1 cartridge body, 10 dynamic mixers



Panasil® monophasic medium

11881 Panasil® monophasic medium
 Refill pack for Plug & Press® Dispenser
 2 x 362mL



Accessories

14101 Panasil® adhesive, 10mL



17253 Dynamic mixers, 50 pieces



17744 Cartridge body



17234 Mixing tips, teal bag of 50



17221 Intraoral tips, clear bag of 96



www.kettenbach.com/www.kettenbachusa.com

Canadian orders, please contact Cerum at 1-800-661-9567

For U.S. orders buy direct from Kettenbach at:

7777 Center Avenue, Suite 280 • Huntington Beach, CA 92647

Toll free fax: 1-888-892-9820 • Local fax: 714-891-7860

Toll free phone: **1-877-532-2123** • Local phone: 714-892-9820